
HUMANTECHNIK

lisa  
signolux

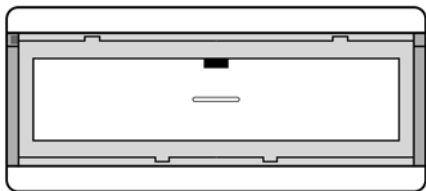**Doorbell-Pushbutton**

**DE**

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The *Lisa-Signolux* »Doorbell-pushbutton« helps people with hearing problems to compensate for their disability. It enables the perception of acoustic events in the home through visible signals.

The »Doorbell-pushbutton« is part of a medical system and is compatible with all devices of the *Lisa-Signolux* wireless system.

## **Package contents**

Please check if all following parts are included:

- Doorbell-pushbutton
- Battery (CR2032, already inserted)
- Double-sided adhesive strip
- User manual
- Warranty card

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.

## **Immediate Safety instructions**

- Please read these instructions carefully and completely before setting up and using the appliance. Be sure to observe the other warnings and safety instructions on page 23 and page 24.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Every main user must have access to it.
- Keep *Lisa-Signolux* devices out of the reach of children! This packaging contains small parts - there is a risk of choking and suffocation!

- The flashes of light emitted by the receiver devices can trigger epileptic seizures in people with photo sensitive epilepsy (PSE).

## Function

A *Lisa-Signolux* alerting system consists of at least one transmitter (e.g. doorbell-pushbutton) and a receiver. If e.g. the doorbell pushbutton is pressed, a radio signal (868,425 MHz radio frequency) is transmitted to the *Lisa-Signolux* receiver. Depending on the receiver type, the received signals are indicated with light, sound or vibration.

## Setup

The battery is already inserted in the device by the manufacturer. After removing the transport securing strip under the clear plastic front, the doorbell-pushbutton is immediately ready for operation.

## Automatic pairing

*Lisa-Signolux* devices can be used together immediately and without prior pairing when delivered.

By default, the doorbell-pushbutton will be indicated by receiver devices with the event icon »Doorbell 2«.

### Important note 1:

This transmitter can be displayed with an event icon of your own choice, this can be done as described in the chapter „*Pairing and assigning to event icons*“.

**However, this causes the according receiver to lose the automatic pairing function and every transmitter previously indicated on the receiver must then be individually paired!**

Important note 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Lisa-Signolux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see Pairing and assigning to event icons).

### **Pairing and assigning to event icons**

The doorbell-pushbutton can be displayed with an event icon of your own choice.

To do this, press and hold the function key of the respective receiver for a few seconds. The event icons of the receiver light up one after the other. Release the function key as soon as the desired icon is lit up.

Now press the doorbell-pushbutton.

Wait for approx. 20 seconds and then test the setup by pressing the doorbell-pushbutton again.


The receiver should now react, showing the according icon.

Up to 8 transmitters can be registered per icon.

## Replacing the battery

In order to be able to insert the coin cell battery (type CR 2032) you first have to open the battery compartment.

a) Place the doorbell-pushbutton in such a way on the work surface that the pivots of the transparent plastic face are at the rear (fig. 1 A and C).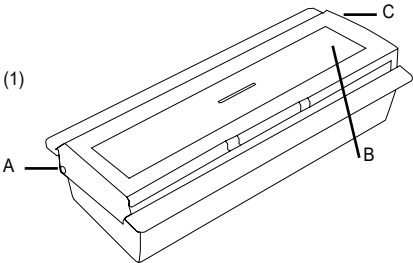


b) Using a pointed object (e.g. biro) carefully press now the small pin on the left side (fig. 1 A).

c) Now you should be able to push up and remove the transparent plastic face (fig. 1 A).

d) You can now open the small screw (fig. 1 B) using a small Phillips screwdriver and remove the front of the doorbell-pushbutton behind which there is the PCB with the battery.

e) The battery is now accessible.

f) If you wish to replace the battery, remove the old one with a small flat screwdriver from its plastic support (fig. 2 D) by lifting the edge slightly (fig. 2 E). Afterwards you can insert a new battery. Kindly ensure the correct polarity of the battery.

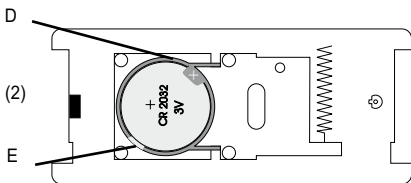### **Inserting the name plate**

a) If you have not opened the housing yet, kindly proceed as described under step 1 a) to c).

b) The transparent plastic face consists of two parts. The labelled name plate can be inserted between them (dimensions: 94 x 24 mm). Please ensure that the labelling is the right way up. Also keep in mind that the name plate has a small cutout for the status LED.

c) Put the two transparent parts of the plastic face back together so that the notches interlock.

d) Please refer to the next chapter for the reassembly of the doorbell-pushbutton.

## **Reassembling the doorbell-pushbutton**

- a) If you have changed only the name tag, proceed to step e).
- b) Before assembling ensure that the small screw (fig. 1 B) can grip at the right spot. If required turn one part of the housing by 180°. Please make sure that the rubber gasket sits correctly within the housing.
- c) Insert the front face into the housing of the doorbell push button and press both firmly together until the housing parts lock in place.
- d) Carefully drive in the small screw (fig. 1 B) with a small Phillips screwdriver.
- e) Now hold the doorbell pushbutton in such a way that the small screw (fig. 1 B) is on the right.
- f) At the right side of the pushbutton there is a small opening (fig. 1 C) where the hinge pin of the transparent plastic face can be inserted.
- g) You can now position the transparent plastic face in such a way until it has almost reached its final position. You only have to carefully press the small pin (fig. 1 A) and lock the transparent plastic face in place.

## **Mounting the doorbell-pushbutton**

You can mount the doorbell-pushbutton using the supplied strips of double-sided tape. Take care to adhere it to a smooth, dry and clean surface.

### Important note:

Kindly keep in mind that metal parts and other building materials may interfere with the transmission of the radio signal.

The device is protected against dust and splash water (IP 54). However, if the push-button is installed outdoors, it should be placed under a roof for protection.

**For your safety:** To ensure the function of the system, a function test must be carried out after installation. If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

## Specifications

 Working range: 10 to 30°C, optimum ambient temperature: 25°C

 Operation: < 90%, non-condensing  
(transport, storage and operation)

Protection class: **IP 54**

Radio frequency: 868,425 MHz

Range: up to 200 m (open space)

Transmission power: <10 mW

Battery type: CR2032 (3 V)

Electrical safety: Low-voltage device

Battery service life: 18 months/battery  
(5 activations per day)

Dimensions (HxWxD): 41 x 90 x 23 mm

Weight: 41 g

Expected service life of the device: approx. 10 years

## Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

### Safety instructions for batteries

- Only use the specified battery type CR2032! There is a risk of explosion if the wrong type of battery is used.
- Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.  
Disposing of batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
- Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
- Remove the battery from the unit if it will be out of use for a longer period of time (> 3 months).
- If the battery condition is insufficient, event messages from the system may be missed!
- Batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets.  
In case of emergency, seek appropriate medical attention immediately.

### Information on product safety

Please observe the following instructions to avoid damage to the unit and possible invalidation of the warranty.

- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- Dropping the unit on a hard surface can damage or destroy it.

## **Maintenance and cleaning**

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants. Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive. No liquid or moisture may penetrate the openings of the unit.

## **Warranty**

The doorbell-pushbutton is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so.

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the serial number in any event.**

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

Help to protect the environment and health of your fellow human beings by the correct disposal of this product. Environment and health are endangered by incorrect disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

### **Disposal of batteries**

The included battery is recyclable. Please dispose of the battery by using a dedicated battery container or returning it to the specialist shop. Please dispose only discharged batteries to ensure environmental protection.

## Compatible *Lisa-Signolux* receiver devices


This transmitter is compatible with all receivers of the *Lisa-Signolux* product range:

- Receiver S (A-2616-W)
- Receiver T (A-2617-0)
- Portable vibrating receiver (A-2619-0)
- Receiver alarm clock (A-2634-0 / W)
- Flashing lamp (A-2644-W)
- Flashing lamp with Type F (CEE 7/3) socket (A-2645-0)
- Receiver T Connect (A-2676-0)

## Markings

 This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.

 This is the marking for the product identification number to identify the medical device.
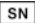
 This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.

 This is the identification for the manufacturer's item number.

 This is the marking for manufacturer of the medical device.

 This symbol indicates the production date of the medical device.

 This is the marking of the country of production of the medical device. This is shown on the device and the packaging.

 This symbol indicates distributors for the medical device.

 This symbol indicates that the user should read these operating instructions attentively.

 This sign indicates warnings that are important for the user as well as important information on use and operational safety.

Temperature range for transport and storage: -5°C to 40°C  
Temperature range for operation: 10°C to 30°C, i optimum ambient temperature: 25°C

Humidity during transport and storage: < 90% (non-condensing)  
Operating humidity: 15% to 90% (non-condensing)

Atmospheric pressure range during transport, storage and operation:  
700 hpa to 1060 hpa

This mark indicates that this appliance does not require sterilisation.

The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on [www.humantechnik.com](http://www.humantechnik.com) in the service area.

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from:  
Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative:  
HUMANTECHNIK GmbH, Im Wörth 25, D-79576 Weil am Rhein, Germany

**All serious incidents involving this product must be reported to the manufacturer and the according local authority!**

Technical specifications may be subject to change without prior notice.

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**Humantechnik Finland Oy**