HUMANTECHNIK

*lisa*

**DE**

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Seite 2

Funk-Blitzlampe

**EN**

**Operating Instructions**

Page 8

RF flash lamp

**FR**

**Mode d'emploi**

Page 14

Lampe-flash radio *lisa*

**NL**

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Pagina 20

Draadloze flitslamp

**IT**

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Pagina 26


Radiolampada flash

**ES**

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
Página 32

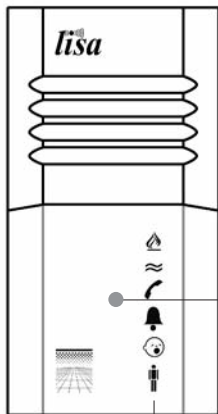Radiolámpara de destellos


Blitz-  
fläche

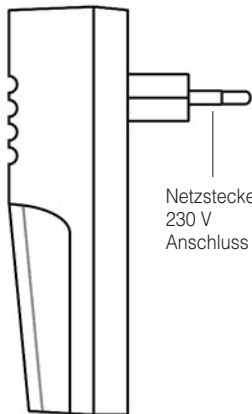Signallämpchen

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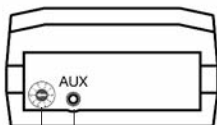Blitzrhythmus  A timing diagram showing a sequence of events: a light pulse (B) followed by a 1.4-second pause (P14), another light pulse (B), a 3.0-second pause (P30), and finally a long horizontal line (W) representing a continuous light state.

**Türklingel I:** Tür-LED leuchtet

Blitzrhythmus  A timing diagram showing a sequence of four light pulses (B) separated by 1.4-second pauses (P14). After the fourth pulse, there is a 4.0-second pause (P40), followed by a long horizontal line (W) representing a continuous light state.

**Türklingel II:** Tür-LED blinkt

Blitzrhythmus  A timing diagram showing a sequence of four light pulses (B) separated by 2.0-second pauses (P20). After the fourth pulse, there is a 4.0-second pause (P40), followed by a long horizontal line (W) representing a continuous light state.

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Blitzrhythmus  A timing diagram showing a sequence of two light pulses (B) separated by a 3.5-second pause (P35). After the second pulse, there is a 4.0-second pause (P40), followed by a long horizontal line (W) representing a continuous light state.

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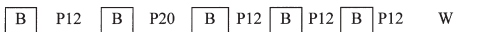
Blitzrhythmus  A timing diagram showing a sequence of two light pulses (B) separated by a 1.2-second pause (P12). After the second pulse, there is a long horizontal line (W) representing a continuous light state.

**Alarm:** Alle LED's blinken

Blitzrhythmus  A timing diagram showing a sequence of six light pulses (B) separated by alternating 2.0-second (P20) and 1.4-second (P14) pauses. The sequence ends with a 1.4-second pause (P14) and does not have a final continuous light state.

**Wasser-Alarm:** Wasser-Alarm LED leuchtet

Blitzrhythmus  A timing diagram showing a sequence of five light pulses (B) separated by alternating 1.2-second (P12) and 2.0-second (P20) pauses. The sequence ends with a 1.2-second pause (P12), followed by a long horizontal line (W) representing a continuous light state.

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Blitzrhythmus  A timing diagram showing a sequence of four light pulses (B) separated by alternating 1.2-second (P12) and 2.0-second (P20) pauses. The sequence ends with a 2.0-second pause (P20), followed by a long horizontal line (W) representing a continuous light state.

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
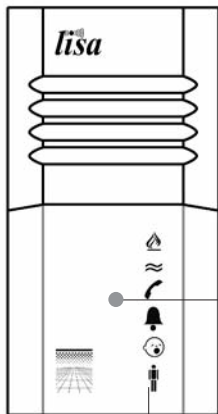
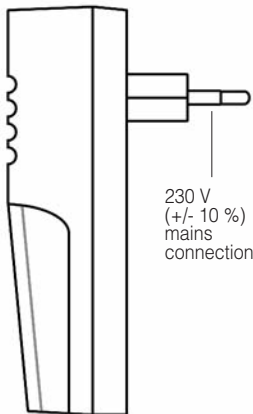
LED displays for  
*lisa* signals

230 V  
(+/- 10 %)  
mains  
connection

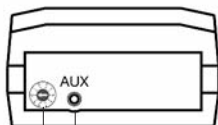Socket for vibrating pad and modules (special accessory)

Channel selector switch

Congratulations on purchasing your cordless *RF flash lamp*. We hope it will provide you with many years of service. This is a proven and reliable product for people with hearing problems and helps to perceive acoustic events in the home through visible alerts.

Please read through the operating instructions carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

### Standard components

Please check if all following components are included:

- *RF flash lamp*
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

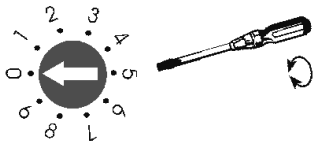
### Functional principle

The *RF flash lamp* receives signals from various transmitters and converts them into light signals. The *RF flash lamp* has a powerful flash tube which alerts you to a signal and LEDs displaying the type of signal being received.

## Setting up the system

### Setting the correct radio channel

A special digital transmission process ensures that the signals are picked up without any errors. This process also enables 10 different radio channels to be set. The default setting for the cordless *RF flash lamp* is channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Using a small screw driver, you can set the channel number you would like to use.

### Installing the cordless *RF flash lamp*

The various transmission signals are converted into different sequences of flashes and signal displays by the cordless *RF flash lamp*:

**Telephone:** Telephone LED lights  
flashing sequence 

B	P14	B	P30	W
---	-----	---	-----	---

**Doorbell I:** Doorbell LED lights  
flashing sequence 

B	P14	B	P14	B	P14	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

**Doorbell II:** Doorbell LED flashes  
flashing sequence 

B	P20	B	P20	B	P20	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

**Babymonitor:** Babymonitor LED lights  
flashing sequence 

B	P35	B	P40	W
---	-----	---	-----	---

**Person call:** Person call LED lights  
flashing sequence 

B	P12	B	P12	W
---	-----	---	-----	---

**Alarm:** All LEDs flash  
flashing sequence 

B	P20	B	P20	B	P14	B	P14	B	P14
---	-----	---	-----	---	-----	---	-----	---	-----

**Water alarm:** The water alarm LED lights  
flashing sequence 

B	P12	B	P20	B	P12	B	P12	B	P12	W
---	-----	---	-----	---	-----	---	-----	---	-----	---

**Fire alarm:** Fire alarm LED lights  
flashing sequence 

B	P12	B	P20	B	P12	B	P20	W
---	-----	---	-----	---	-----	---	-----	---

**P12** pause 1,2 seconds

**P30** pause 3,0 seconds

**P14** pause 1,4 seconds

**P35** pause 3,5 seconds

**P20** pause 2,0 seconds

**P40** pause 4,0 seconds

**B** flash

**W** repeat

The indicator LEDs remain on for 40 seconds.

### **Important note**

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

### **Maintenance and care**

All *lisa* devices are maintenancefree. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Note: remove the *RF flash lamp* from mains socket! Never use spirits, thinners or other organic solvents. Please do not expose *lisa* devices to direct sunlight for a long time and please protect them from excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### **Warranty**

All *lisa* devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

### Important safety notices

- Keep *lisa*-devices out of the reach of children!
- Flashes of light can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

### Technical data

Power supply:	230 V, 50 Hz,
Power consumpt.:	approx. 1 W
Frequency:	868,35 MHz
Height::	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	215 g

This device satisfies the following EC directives:

2011 / 65 / EC + 2015 / 863 / EC RoHS-directive

2012 / 19 / EC WEEE-directive

2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at <http://www.humantechnik.com/service>

Humantechnik's UK authorised representative and UK declaration of conformity available from:

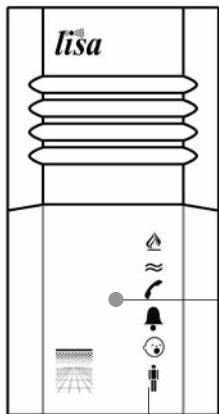
Sarabec Ltd.

15 High Force Road,

MIDDLESBROUGH TS2 1RH


United Kingdom

Technical specifications subject to change without notice.

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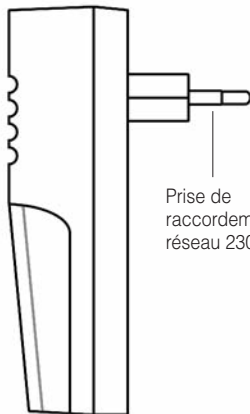
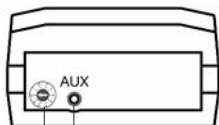
**Telefoone:** De telefoon-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P30', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

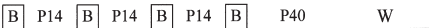**Deurbel I:** De deurbel-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

**Deurbel II:** De deurbel-LED knippert

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

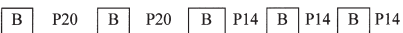
**Babyoproep:** Babyoproep-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P35', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'. The boxes are connected by horizontal lines.


**Personenoproep:** Personenoproep-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

**Alarm:** Alle LED's knippen

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14'. The boxes are connected by horizontal lines.

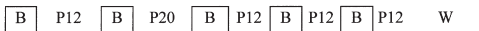**Wateralarm:** Wateralarm-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

**Brandalarm:** Brandalarm-LED brandt

Flitsritme  A timing diagram showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', and finally a box labeled 'W'. The boxes are connected by horizontal lines.

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**Humantechnik GmbH**Im Wörth 25
**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road