



# vac-clean uv 2<sup>®</sup>

Mini vacuum cleaner for hearing aids and hearables.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 3
DE	GEBRAUCHSANWEISUNG	4 - 5
FR	MODE D'EMPLOI	6 - 7
ES	MANUAL DE INSTRUCCIONES	8 - 9
PT	MANUAL DE INSTRUÇÕES	10 - 11
IT	ISTRUZIONI PER L'USO	12 - 13
NL	GEBRUIKSAANWIJZING	14 - 15
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	16 - 17
TR	KULLANIM TALİMATI	18 - 19
DK	BRUGERVEJLEDNING	20 - 21
SE	BRUKSANVISNING	22 - 23
NO	BRUKSINSTRUKSJONER	24 - 25
PL	INSTRUKCJA OBSŁUGI	26 - 27
CN	使用说明 (简体中文)	28 - 29
JP	取扱説明書	30 - 31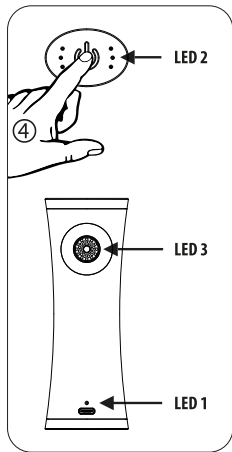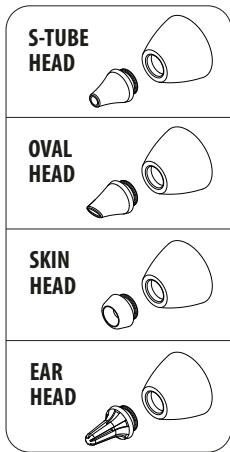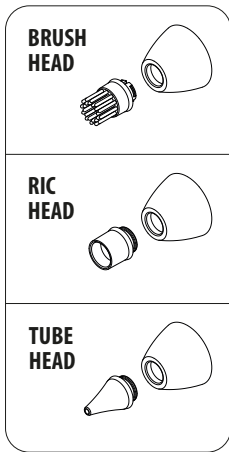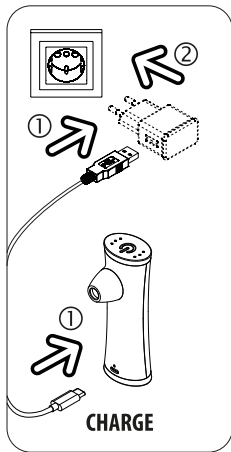

## EN INSTRUCTION FOR USE

### Initial operation

1. Plug the USB cable into a USB power supply (5V/1A) and the USB-C plug into the USB-C socket of your vac-clean uv 2®.
2. Then plug the power supply unit into a mains socket. ⓘ LED 1 flashes green
  - The vac-clean uv 2® is charged in approx. 2.5 hours and runs for approx. 2.5 hours. ⓘ LED 1 lights up green

### Operation with the attachment head

3. Insert the attachment head suitable for your cleaning purpose into the base attachment.
4. Briefly touch the I/O button and the vacuum motor starts at the strongest suction level (ooo).
  - ⓘ LED 2 lights up (ooo)
    - The blue LED light starts automatically for better illumination of the areas to be cleaned.
    - ⓘ LED 3 lights up
      - Move the vac-clean uv 2® with light pressure over all areas to be cleaned and vacuum up the dirt.
5. Once cleaning is complete, switch the appliance off by touching the I/O button for at least 3 seconds. Otherwise the device will automatically switch-off after 4 minutes.
  - ⓘ all LEDs turn off

## **Maintenance and care**

To clean your vac-clean uv 2<sup>®</sup>, we recommend wiping it with damp cleaning cloths. Unscrew the base attachment with the attachment head after use and clean it in an ultrasonic bath, e.g. u-sonic 3. Then dry well under a drying hood, e.g. dry-cap uv 3.1<sup>®</sup>. Wipe the filter and LED 3. Please also observe the care and operating instructions for your hearing aids. Hearing aids must be switched off before cleaning. The vac-clean uv 2<sup>®</sup> is very energy efficient. The lithium-ion battery only needs to be recharged when it is empty (see Initial operation) ⓘ LED 1 lights up red. Your vac-clean uv 2<sup>®</sup> has 3 suction levels (ooo / oo / o) with different strengths. By briefly touching the I/O button, you can switch between the suction levels as required during operation.

## **! Warnings**

Avoid contact with water! The vac-clean uv 2<sup>®</sup> must not be operated near washbasins or similar! Avoid mechanical stress caused by impact and falling! The vac-clean uv 2<sup>®</sup> must not be operated in the vicinity of fire or easily flammable objects! In the event of malfunctions, return the appliance to the dealer! Do not disassemble the appliance! Children must be supervised to ensure that they do not use the appliance as a toy! This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and/or knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Caution is advised for all short-term applications on the body; consult a doctor in the event of abnormalities.

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© 2010-2024 flow-med GmbH. All rights reserved.
**flow-med GmbH**
Otoplastic AG