HUMANTECHNIK

lisa  
signolux

**Receiver Alarm Clock**

**DE**

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Seite 4

**EN**

**User manual**

Page 18

**FR**

**Mode d'emploi**

Page 32

**NL**

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Pagina 46

**FI**


**Käyttöohje**

Sivu 60

**DA**

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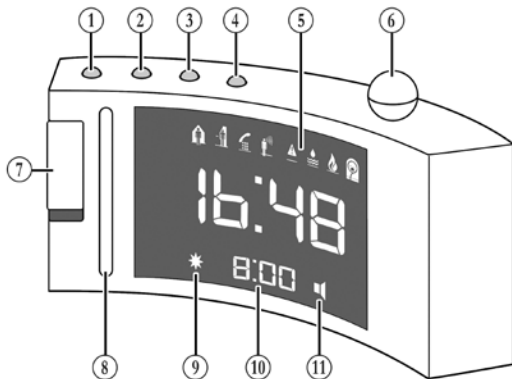Side 74

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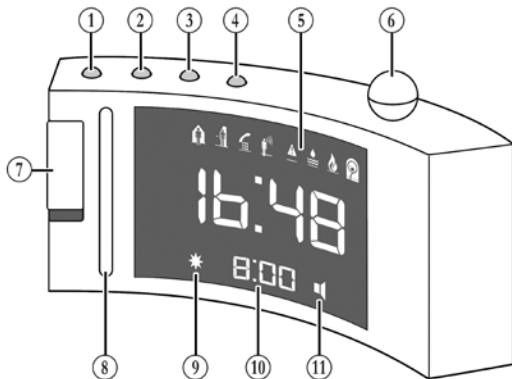
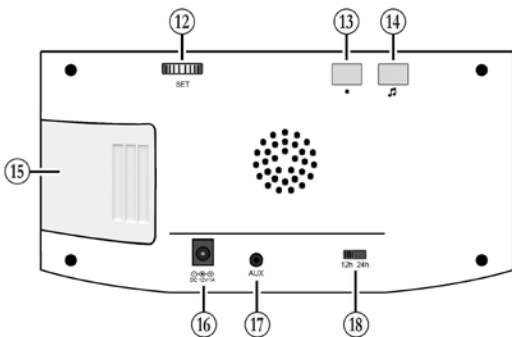
14 - Melody button

17 - „AUX“-connector

The *Lisa-Signolux* »Receiver Alarm Clock« helps people with hearing problems to cope with everyday life. It enables the perception of acoustic events in the home through visible signals.

The »Receiver Alarm Clock« is part of a medical system and compatible with all devices of the *Lisa-Signolux* alerting system.

## Scope of Delivery

Kindly check if all following parts are included:

- »Receiver Alarm Clock«
- Power supply
- User manual
- Warranty card

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.

## Immediate safety instructions

- Before commissioning and using the device, read this manual carefully and completely. Observe the other warnings and safety instructions on page 26 and 27.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Each main user must be able to access it.
- Keep *Lisa-Signolux*-devices out of the reach of children! This packaging may contain small parts which may present a choking hazard!
- The flashes of light emitted by the »Receiver Alarm Clock« can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

## Function

The »Receiver Alarm Clock« is an electronic clock with a digital time display and a complete *Lisa-Signolux* receiver, capable of displaying the alerts of all *Lisa-Signolux* transmitters. A received alert is displayed by flash and/or sound and can be identified by 8 illuminated event icons.

The following accessories can be connected to the »AUX« connector (17):

- vibrating pad with batteries A-3303-0
- vibrating pad without batteries A-3304-0
- vibrating contraption A-3310-0

## Setup

Connect the power supply unit to the 12 V DC socket (16) of the alarm clock and plug it into a 230 V power outlet. After a few seconds, the alarm clock is functional.

### Setting the time

To set the time, press the time set button »  « (1) for one second. The hour digits will start to flash. Set the desired hour by turning the adjusting wheel (on the reverse side of the of the clock) to the left or right.

Press the time set button »  « again. The minute digits will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the time set button »  « again will end the time setting procedure and the set time is displayed.

### Setting the alarm time

To set the alarm time, press the alarm time button »  « (2) for one second. The hour digits of the alarm time will start to flash. Set the desired hour by turning the adjusting wheel to the left or right.

Press the alarm time button »  « again. The minute digits of the alarm time will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the alarm time button »  « again will end the alarm time setting procedure and the set alarm time is displayed.

### Alarm ON / OFF

The waking alarm can be switched on and off with the slider switch (7) on the left side of the alarm clock.

Slider in upper position (green area is visible): Alarm is switched ON.

Slider in lower position: Alarm is switched OFF.

### Alarm tone ON / OFF

Press the alarm tone button »  « (4) for one second to switch the alarm tone ON or OFF. If the alarm tone is on, the speaker symbol (11) will be visible in the display.

## Setting the alarm tone volume

Press and hold the alarm tone button »**«** while adjusting the volume of the alarm tone. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone volume. Release the alarm tone button to end the volume setting procedure.

## Setting the alarm tone pitch

While adjusting the alarm tone pitch, press and hold both the alarm tone button »**«** and the spherical snooze button (6) at the same time. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button and the snooze button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone pitch. Release the buttons to end the pitch setting procedure.

## Flash ON / OFF

Press and hold the flash button »**«** (3) for one second to switch the flash ON or OFF. If the flash is on, the flash symbol (9) will be visible in the display.

## Setting the nighttime display brightness

The brightness of the displayed digits is automatically adjusted according to the light conditions within the room. During the day, the digits are displayed with full brightness. In darkness, the set nighttime display brightness is activated.

To set the nighttime display brightness, press and hold the flash button »**«**. Keep the flash button pressed and turn the adjusting wheel to the left or right to adjust the desired display brightness for the night. Release the button to end the setting procedure.

If this adjustment is done during daylight conditions, the display will switch back to full brightness after a few seconds.

## Deactivating the illuminated digits

If the illuminated digits are too bright for you at night, even at their darkest setting, they can also be deactivated completely. This has no influence on the indication of events!

Digits off: Press and hold the buttons »**«** and »**«** simultaneously for 5 seconds. For confirmation, the digits flash 2 times and then switch off after 60 seconds.

Digits on: Press and hold the buttons »« and »« simultaneously for 5 seconds. For confirmation, the digits flash 4 times and then switch on again immediately.

The luminous digits can be activated for 60 seconds at any time by pressing any key.

## **Snooze function**

You can interrupt a current alarm by pressing on the spherical snooze button. The alarm will be repeated after 5 minutes.

## **12 / 24 hour mode**

The time can be displayed in a 12- or 24 hour mode. The adjustment is made with the »**12h/24h**« switch (18) on the back of the alarm clock.

## **Power cut backup**

The battery compartment (15) can hold 4 batteries of size AAA (not included). In the event of a power cut, these guarantee the continuation of the device settings, the time and the display of the *Lisa-Signolux* alerts for about 1 day.

**Please note:** During a power cut, the digits and the flash are not active as the power consumption is too high for the backup batteries! The batteries must be removed when the alarm clock is not in use (with the mains adapter unplugged), otherwise they will be discharged after 1 day.

## **Automatic pairing**

*Lisa-Signolux* devices can be used together immediately and without prior pairing as delivered.

Which event icon is displayed depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

## **Optional manual pairing**

**Note 1:** If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter „*Linking transmitter alerts with event icons*“.

However, this causes the »Receiver Alarm Clock« to lose the automatic pairing function and all transmitters previously shown on the alarm clock must be individually paired.

**Note 2:** Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Signolux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see „*Linking transmitter alerts with event icons*“).

### **Reset to factory defaults**

Delete the pairing of each event icon (see „*Deleting transmitters*“). When all previously assigned transmitters have been removed from the memory, the preset transmitter indication is restored and the transmitters are automatically displayed again.

### **Linking transmitter alerts with event icons**

Up to 8 transmitters (e.g. the doorbell push-button) can be assigned to each event icon. This assignment can be adapted by pairing.

Please keep in mind that a transmitter can only be paired if it is not yet known in the memory of the *Lisa-Sinolux* receiver. If required, you might first have to delete the transmitter from the memory.

- a) Keep the function button »●« (14) pressed for some time. The event icons light up in sequence. Release the button as soon as you see the desired icon.
- b) Now trigger the signal using your transmitter (e.g. operate the doorbell pushbutton).
- c) Wait for about 20 seconds and test the function.

### **Deleting transmitters**

Do the following if you want to remove transmitter from the memory:

- a) Keep the function button »●« pressed for some time. The event icons light up in sequence. Let go of the button as soon as you see the desired icon.
- b) Now press the function button »●« and the melody button »🎵« until all icons briefly light up.

Please note that this will remove all assigned transmitters from the selected event icon.

## Setting the melody

You can choose the melody / sound effect for each event icon:

- First trigger a signal (e.g. operate the doorbell pushbutton).
- The icon assigned to the transmitter should light up and the previously chosen melody should be heard.
- Press and hold the melody setting button for 2 seconds. The icon starts to flash.
- Repeatedly press the melody setting button »« until you have found the desired melody / sound effect (you can choose from 15 different sounds).
- Once the icon ceases to flash, the selected melody has been assigned.

## Event icon description

The event icons stand for:

Doorbell 1 green light

Doorbell 2 green light

Telephone orange light

Person call yellow light

Alarm red light

Water alarm blue light

Fire alarm red light

Baby white light

## Recalling the last alert, cancelling the current alert

Press the »« button to display the last registered alert again.

You can also cancel an active indicated alert by pressing this button.

## Connecting accessories

The „AUX“ -connector (17) is intended for the connection of vibrating pads or the vibrating contraption.

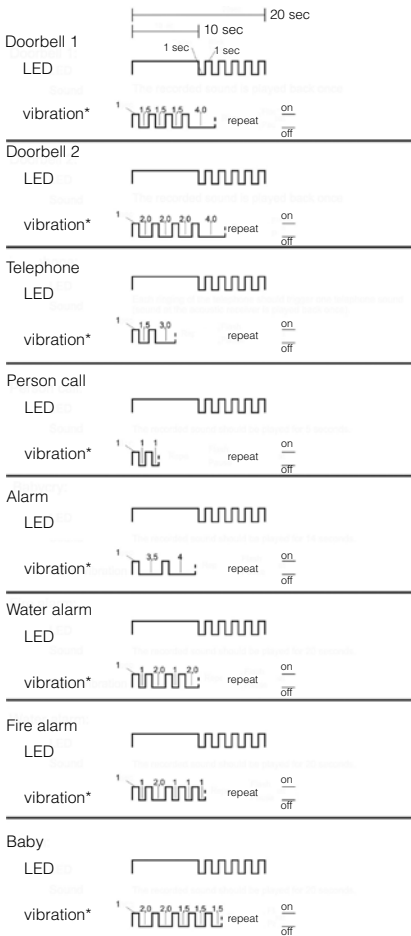
Certified accessories:

- Vibrating pad with batteries (A-3303-0)
- Vibrating pad without batteries (A-3304-0 / A-3304-W)
- Vibrating contraption (A-3310-0)

## Signal distinction

\* only with attached vibrating pad

## Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

### **Safety instructions for rechargeable batteries and power supply**

- Only use the specified battery type NiMH AAA (HR03)! There is a risk of explosion if the wrong type of battery is used.
- Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.
- Disposing of batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
- Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
- Use only the original power supply with the receiver.
- Remove the batteries from the unit if it will be out of use for a longer period of time (> 3 months).
- If the batteries are depleted, event messages from the system may be missed!
- Lay the cable of the power supply unit in such a way as to avoid tripping hazards and possible damage to the cable.
- Rechargeable batteries and regular batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

### **Information on product safety**

- Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.
- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- Only place the unit on flat surfaces. Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.
- Do not disassemble the device or the power supply unit; there is a risk of electric shock. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humantechnik.

- Do not place the device in extremely bright locations, such as sunlit windowsills, as there is a risk of not being able to see the flash.
- This unit is not suitable for outdoor use. Do not allow the unit to become damp or wet.
- Only use units of the *Lisa-Signolux* system where electronic devices are permitted.
- When using in aircraft, it is essential to obtain the prior consent of the flight personnel.
- Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to *Lisa-Signolux* devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

### **Environmental conditions**

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

## Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants. Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive. No liquid or moisture may penetrate the openings of the unit.

## Service, warranty and warranty conditions

The »Receiver Alarm Clock« has a high level of operational safety. If faults should occur despite proper operation, please contact your specialist dealer or contact Humantechnik GmbH directly.

The warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Likewise, damage due to natural forces, such as fire and flooding, is not covered by the warranty. Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

## Always specify the serial number in any event.

### Battery disposal

Please dispose of used rechargeable batteries and batteries properly, according to your local regulations.

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

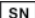
Help to protect the environment and health of your fellow human beings by the correct disposal of this product. Environment and health are endangered by incorrect disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

## Markings

 This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.

 This is the marking for the product identification number to identify the medical device.

 This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.

 This is the identification for the manufacturer's item number.

This is the marking for manufacturer of the medical device.

This symbol indicates the production date of the medical device.

This is the marking of the country of production of the medical device. This is shown on the device and the packaging.

This symbol indicates distributors for the medical device.

This symbol indicates that the user should read these operating instructions attentively.

This sign indicates warnings that are important for the user as well as important information on use and operational safety.

Temperature range for transport and storage: -5°C to 40°C  
Temperature range for operation: 10°C to 30°C, i optimum ambient temperature: 25°C

Humidity during transport and storage: < 90% (non-condensing)  
Operating humidity: 15% to 90% (non-condensing)

Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa

This mark indicates that this appliance does not require sterilisation.

The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com) in the service area.

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from:

Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative: HUMANTECHNIK GmbH, Im Wörth 25, D-79576 Weil am Rhein, Germany

**All serious incidents must be reported to the manufacturer and the according local authority!**

## Specifications

 Working range:	10 to 30°C, optimum ambient temperature: 25°C
 Operation:	< 90%, non-condensing (transport, storage and operation)
Protection class:	<b>IP 20</b>
Electrical safety:	Protection class 2
Reception frequency:	868,425 MHz
Transmission range:	up to 200 m in open spaces, up to 80 m within buildings
Power supply:	110 - 240 V AC /12V DC - 1000mA
Height:	103 mm
Width:	212 mm
Depth:	70 mm
Weight:	337 g (without batteries)
Expected service life of the device:	approx. 10 years

Technical specifications may be subject to change without prior notice.

### Compatible *Lisa-Signolux*-transmitters

The device is compatible with all transmitters of the *Lisa-Signolux* product range:

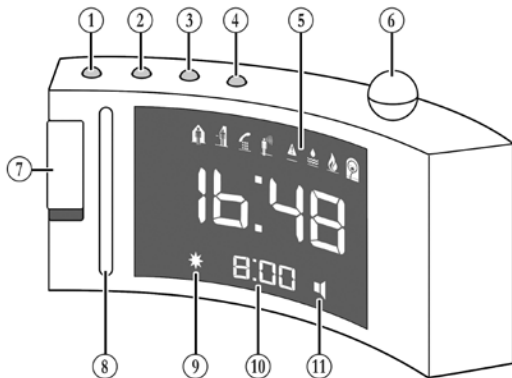
- Direct universal transmitter (A-2655-0 / W)
- Universal acoustic transmitter (A-2656-0 / W)
- Doorbell-pushbutton (A-2657-0 / W)
- Doorbell/Personcall pushbutton (A-2659-0 / W)
- Smoke detector Guardion (A-2638-0)
- Smoke detector Guardion Interlink (A-2642-0)
- Combined detector CO/Temp (A-2672-0)
- Combined detector CO/Temp Interlink (A-2674-0)
- Soundmonitor (A-2680-0 / W)
- Alarmmonitor „Alarmo“ (A-2681-0 / W)

**For your safety:** To ensure the function of the system, a function test must be carried out after installation.

If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

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