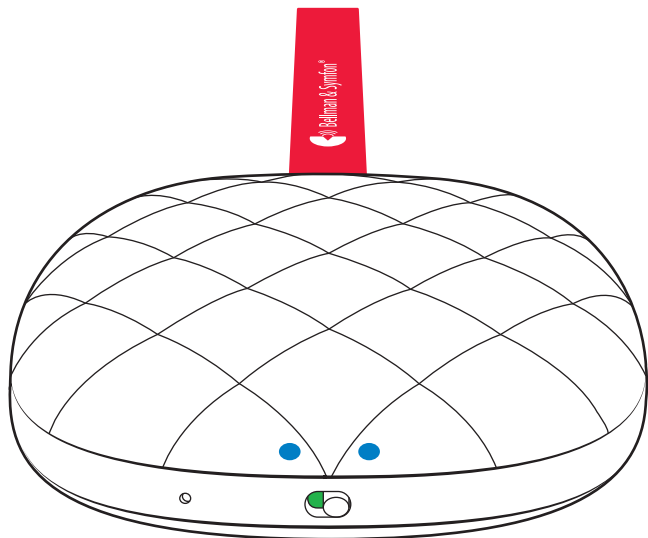
BE1221 VIBIO BED SHAKER

BE1221 Vibio

3	Bedienungsanleitung
7	Brugervejledning
11	User manual
15	Manual de usuario
19	Käyttöopas
23	Mode d'emploi
27	Gebruiksaanwijzing
31	Brukermanual
35	Bruksanvisning
39	ユーザーマニュアル
43	用戶手冊
47	用戶手冊

DE

DK

EN

ES

FI

FR

NL

NO

SE

JP

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Willkommen!

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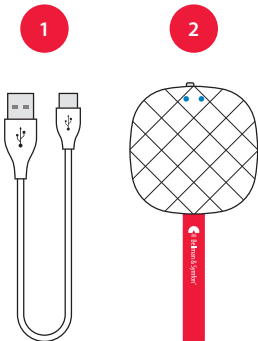
Welcome!

What you'll need

- A mobile phone with iOS 11 / Android 4.4 or later.
- A USB power adapter


What's in the box

- 1 Charging cable
- 2 Vibrio bed shaker

Look for the blinking eyes

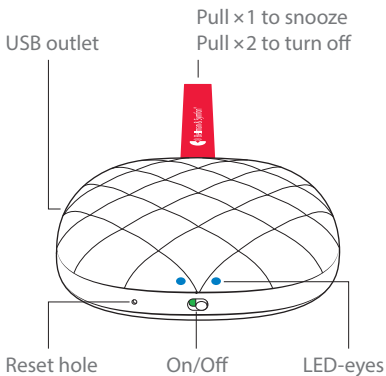- **White** means it's activated by an alarm or communicating with your mobile.
- **Blue** means it's ready for pairing.
- **Green** means it's charging.
- **Yellow** means you have an incoming call or message.
- **Red** means the battery level is low.

Installing Vibio

1 Power it up

Connect the charging cable to your Vibio and plug the connector into a USB power adapter or a USB computer outlet.

2 Turn it on

Slide the On/Off switch to the right and keep Vibio close to your mobile.

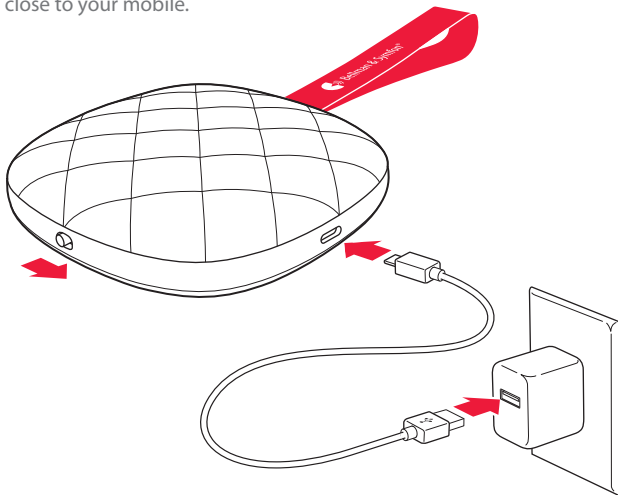
3 Install the app

Download the **Vibio** app from the App Store® or Google Play™ and follow the setup instructions.

EN

Using the app

Managing alarms

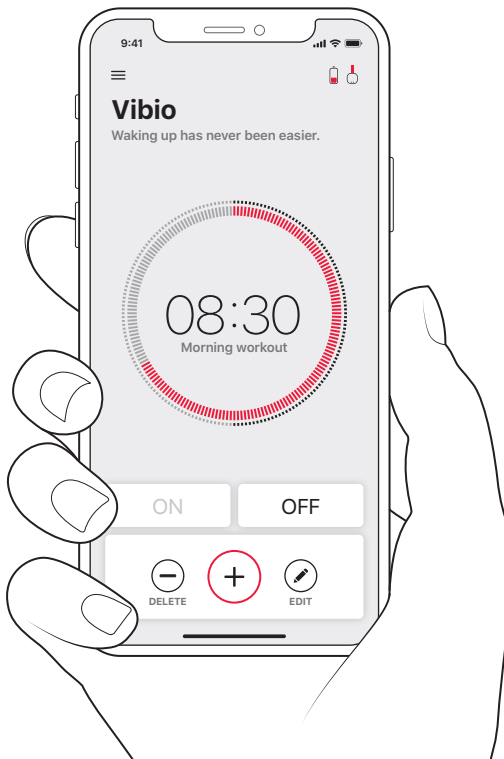
-  **Add.** Tap the plus button to add a new alarm.
-  **Edit.** Tap the pencil button to edit an alarm.
-  **Delete.** Tap the minus button to delete an alarm.
-  **On/Off.** Tap the ON or OFF button to enable or disable an alarm.

Monitoring Vibio

-  **Connection.** The Vibio icon shows the connection status.
-  **Battery.** The battery icon shows the current battery level.

Learning more

-  **Help.** Tap the Menu > Help to learn more about the app.

Troubleshooting

If	Try this
The LEDs are blinking in red	The battery level is low. Charge Vibio using the supplied cable.
The LEDs don't light up	The batteries are depleted. Charge Vibio for about 1.5 hour.
Vibio doesn't connect to my mobile	<p>Start by checking the Bluetooth connection on the app home screen. If the Vibio icon is red and crossed out, try the following:</p> <ol style="list-style-type: none">1 Move within Bluetooth range. Move your mobile closer to Vibio.2 Turn Bluetooth Off and On. Swipe to open the Control Center / Quick Settings Panel on your mobile and tap the Bluetooth icon. <p>If it still doesn't work, you need to reset and reconnect the Vibio bed shaker and app manually. Try the following:</p> <ol style="list-style-type: none">1 Force-quit the app. Note that it's not enough just to close it.2 Disconnect Vibio. Go to Settings > Bluetooth. Tap ⓘ / ⚙️ and Forget This Device to disconnect "Vibio". Make sure Bluetooth is on.3 Reset Vibio. Pull and hold the strap until the LEDs starts to flicker in blue. Release and pull it again until the LEDs starts to breathe.4 Setup Vibio. Start the Vibio app again and follow the instructions.
Intended use Personal vibrating alarm device to be used with a mobile phone application for wakeup.	Dimensions and weight Size: 27 × 94 × 94 mm, 1.1 × 3.7 × 3.7" Weight: 153 g, 5.4 oz.
Power and Battery Battery power: 1 × 3.7 V AA Lithium-ion Power consumption: Active with strong vibration: < 1000 mA, Idle: < 100 μA Operating time: Up to 60 days / charge Charging time: Up to 1.5 hour @ 500mA Charging current: Up to 500 mA, 5V DC	Frequency and coverage Connectivity: Bluetooth Bluetooth frequency: 2402 – 2480 MHz Output power: Max 2.5mW (4dBm) Coverage: Up to 30 m, 33 yd, free field.
Compatibility iOS 11 / Android 4.4 and later, iPhone 5s and later / Modern Android devices	Cables and connections Cable length: 1 m, 39" Contacts: USB type A to USB Micro-B
	Maintenance and cleaning Maintenance free. Clean with a dry cloth. Do not use household cleaners, aerosol sprays, alcohol or abrasives.
	Environmental requirements For indoor use only Temperature: 0°– 35° C, 32°– 95° F Relative humidity: 15% – 90%
	Safety and warranty conditions See separate folder
	Bellman & Symfon Group AB Södra Långebergsgatan 30 436 32 Askim Sweden Phone +46 31 68 28 20 E-mail info@bellman.com
	© and TM 2021, Bellman & Symfon AB, all rights reserved.

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TM and © 2021 Bellman & Symfon AB. All rights reserved.
Bellman & Symfon Asia Ltd
Bellman & Symfon North America Inc.